

# A-dec ICX<sup>®</sup>

## Waterline Treatment Tablets



0.7-Liter bottle



2-Liter Bottle

Convenient single-dispensing packets eliminate measuring, mixing and messy solutions. Available for 0.7-liter and 2-liter bottles.



ICX Tablet

## Simple. Effective. Affordable.

A-dec ICX waterline treatment tablets offer a unique and effective way to maintain clean dental unit waterlines—and it's incredibly easy to use. There's no measuring or mixing, because the effervescent ICX tablet does the work for you.\* Simply add one tablet to an empty self-contained water bottle before each filling. As the tablet dissolves, it releases ingredients into the water that help maintain less than or equal to 10 colony forming units (CFU) per milliliter, preventing the accumulation of odor and foul-tasting bacteria. ICX continues working in the system with each treatment, protecting the waterlines from further contamination and reducing the need for frequent shock treatments. And, since the concentration of ingredients in the water remains very low, ICX won't harm your equipment.

With ICX, you have peace of mind knowing that your waterline maintenance protocol is not only easy and effective, but also safe for your investment.

\*As with all waterline protocols, quality results require adherence to the manufacturer's recommended process.

# A-dec ICX

## Convenient:

- Specially formulated effervescent tablets
- Simple single-dose package; drop tablet into an empty self-contained water bottle and fill

## Effective residual protection:

- Maintains dental unit waterline effluent  $\leq 10$  CFU/ml.
- Eliminates odor and foul tasting bacteria
- Reduces build-up of deposits
- Continuously present in water system; provides a preventive, proactive (vs. reactive) solution
- Remains in water up to two weeks during periods of equipment non-use
- Eliminates the need to purge waterlines at night
- EPA registered
- Peer-reviewed studies on ICX:\*
  - *Journal of Clinical Dentistry*
  - *JCD 2004, Vol. XV, Number 1, pp. 17-21*
  - *Journal of the American Dental Association*
  - *JADA June 2004, Vol. 135, pp. 799-805*

Contact your  
authorized A-dec  
dealer for pricing  
and availability.

## Distribution, packaging and use:

- Distributed exclusively through A-dec's worldwide authorized dealer network
- Tablets are available for two bottle sizes: 0.7- and 2-liter
- Individually foil-wrapped tablet, packaged 50 per box
- Cost effective; low annual cost
- Approximate usage rates for a single treatment room (one 2-liter or one and a half 0.7-liter bottles per day for a 16-day month):
  - *Two months per 50-count box, 0.7-liter tablet*
  - *Three months per 50-count box, 2-liter tablet*

## Safety:\*\*

- Non-corrosive to dental equipment
- No odor or taste
- Includes three primary active ingredients: sodium percarbonate, silver nitrate, and cationic surfactants

## Bonding:

- Compatible for use with restorative dental materials; does not affect bond strength
- Clinical study on dentin bonding:\*
  - *Journal of Clinical Dentistry*
  - *JCD 2004, Vol. XV, Number 1, pp. 28-32*

\* For independent clinical study reprints or additional information, contact A-dec Customer Service at 1.800.547.1883 (1.503.538.7478 worldwide).

\*\* Safety Data Sheet (SDS) are available at a-dec.com.



### A-dec Headquarters

2601 Crestview Drive  
Newberg, Oregon 97132  
United States

Tel: 1.800.547.1883 Within USA/CAN  
Tel: 1.503.538.7478 Outside USA/CAN  
Fax: 1.503.538.0276  
a-dec.com

### A-dec Australia

Tel: 1.800.225.010 within AUS  
Tel: +61.(0)2.8332.4000 outside AUS

### A-dec China

Tel: 400.600.5434 within CN  
Tel: +86.571.89026088 outside CN

### A-dec United Kingdom

Tel: 0800.233285 within UK  
Tel: +44.(0).24.7635.0901 outside UK

A-dec, the A-dec logo, A-dec Inspire, Cascade, Century Plus, Continental, Decade, ICX, ICV, Performer, Preference, Preference Collection, Preference ICC, and Radius are trademarks of A-dec Inc. and are registered in the U.S. and other countries. A-dec 500, A-dec 400, A-dec 300, A-dec 200, and reliablecreativesolutions are also trademarks of A-dec Inc. None of the trademarks or trade names in this document may be reproduced, copied, or manipulated in any manner without the express, written approval of the trademark owner.

Certain touchpad symbols are proprietary to A-dec Inc. Any use of these symbols, in whole or in part, without the express written consent of A-dec Inc. is strictly prohibited.