


A-dec Inspire® 396 Mobile Cart

Product Models and Versions Covered in This Document*


Models	Versions	Description
396	A	Dental Furniture

*Equipment shown in this document may include optional features or accessories. Some product models, versions, and options in this document may not be available in certain regions.

Prepare the Cart

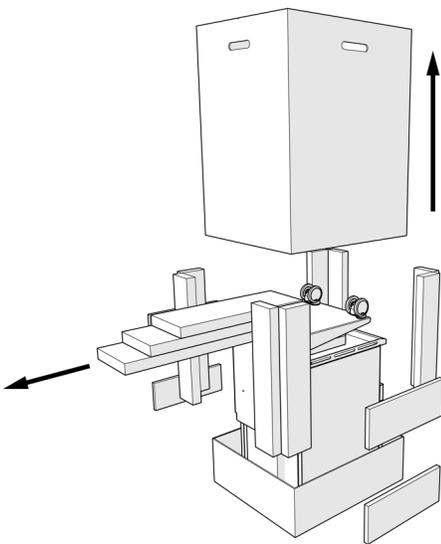


1 Remove the items from the cart.

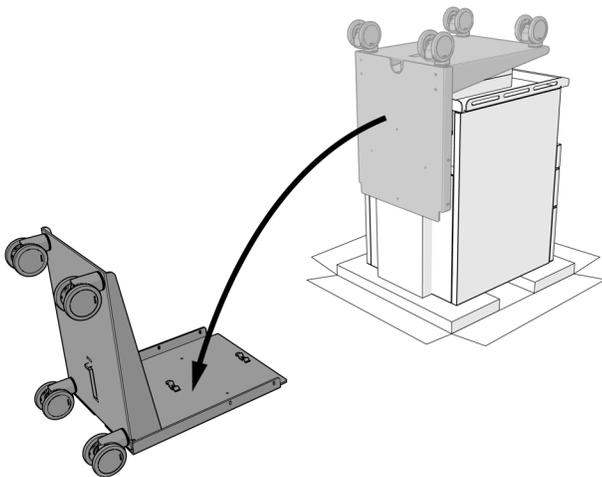


2 Remove the base from the cart.

3 Remove the drawers from the cart.

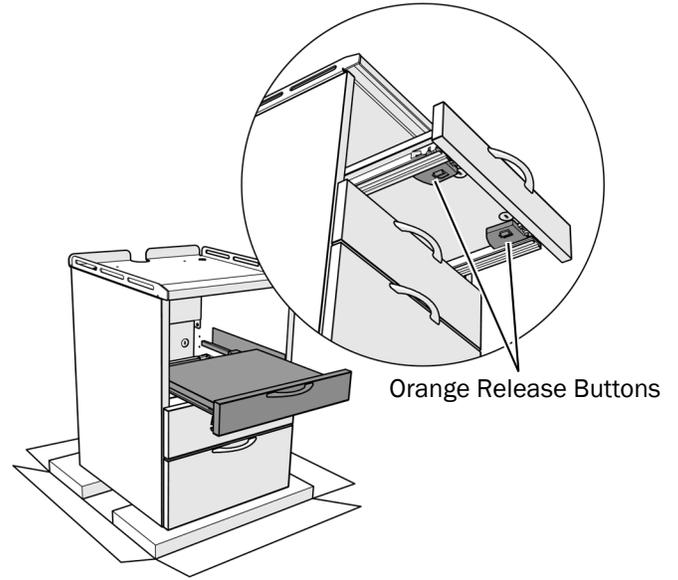

Equipment Drawers

1. Squeeze the large orange release buttons on the bottom of the drawer and pull the drawer out.

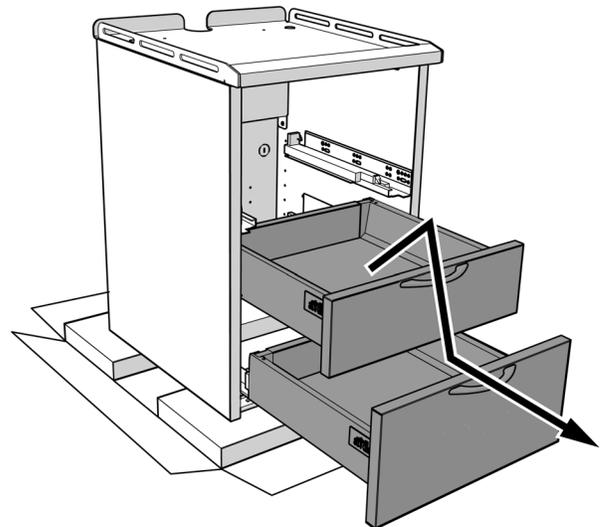


All Other Drawers

1. Pull the drawer out to the stop.
2. Lift the front of the drawer, pull out, then down.


Install the Cart Base

Recommended Tool


- #2 Pozidriv[®] screwdriver

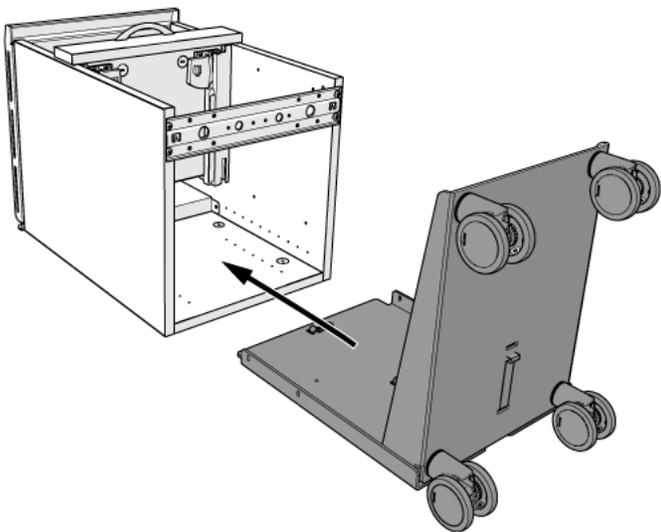

- 1** Place the cart on its back.
- 2** Place the base inside the cart.

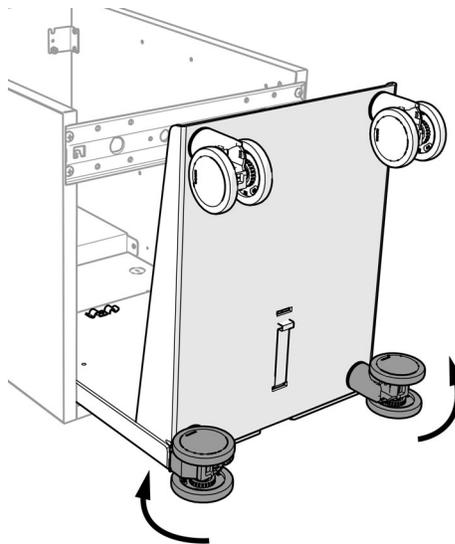

- 3** Rotate the back wheels so that the base lays flat against the cart.

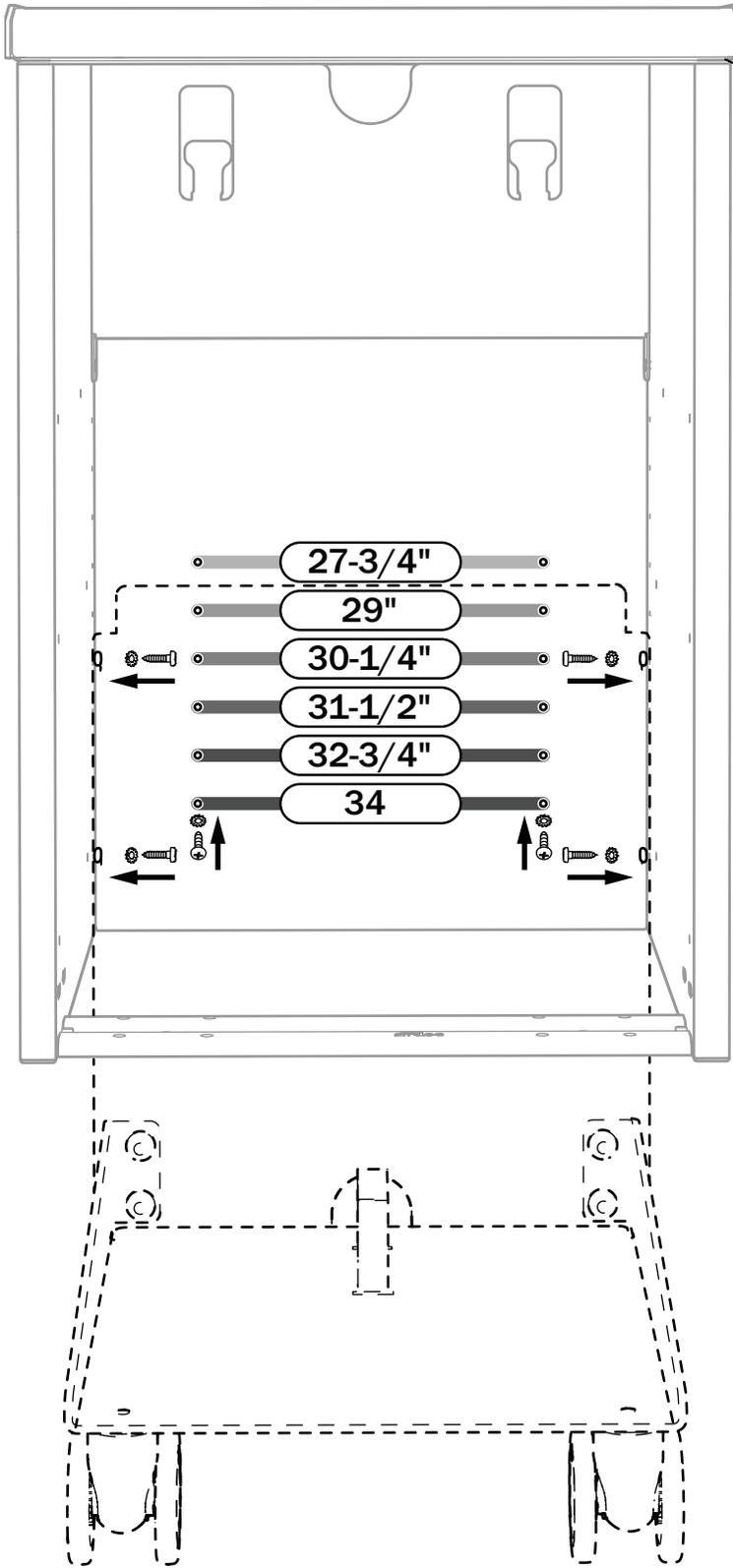

NOTE Dimensions shown in the image represent the height of the cart from the floor to the bottom of the countertop when the base is secured to the corresponding set of holes.


- 4 Align the holes in the back of the cart for the desired height of the countertop.




CAUTION To prevent damage to the cart and ensure a safe installation, always use six screws and washers to secure the base to the cart.

- 5 Install two screws and star washers to secure the back of the base to the back of the cart.
- 6 Install two screws and star washers to each side to secure the base to the sides of the cart.


Install the Drawers

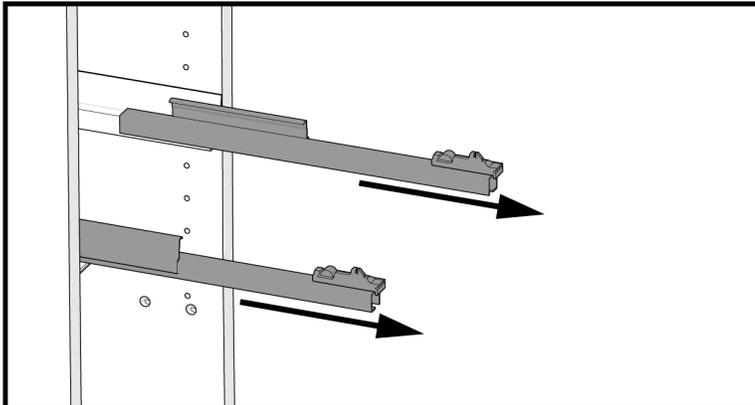

1 Pull out the drawer slides.

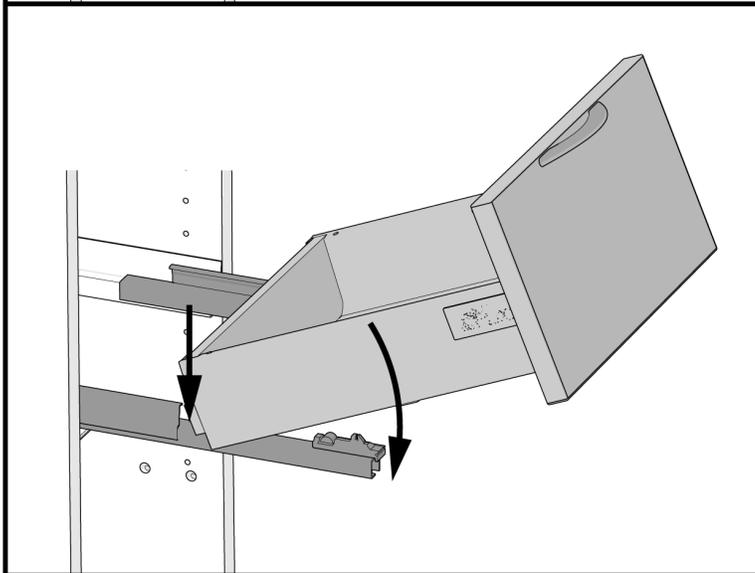

2 Align the drawer with the extended slides then insert the drawer onto the slides.



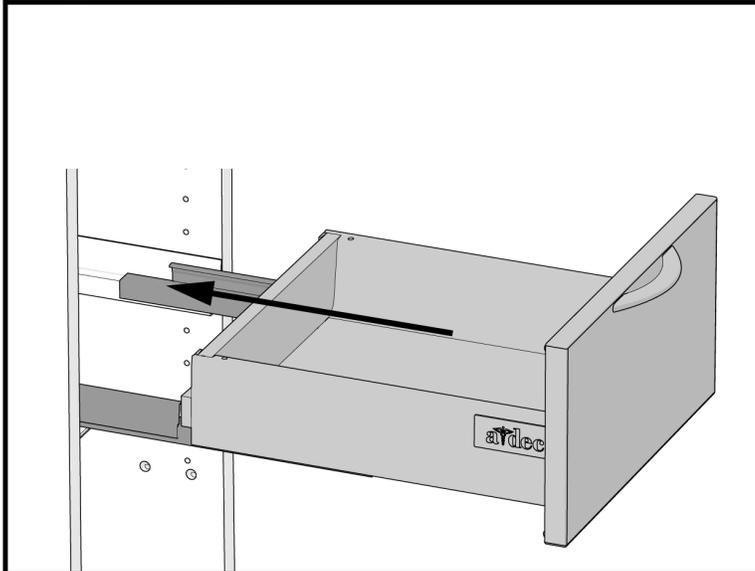


3 Push the drawer in and ensure that it is engaged with both slides. Confirm the drawer moves smoothly through its full range of motion.


Adjust the Drawers

Complete the procedure for the type of drawer to adjust the face of the drawers.

Equipment Drawers

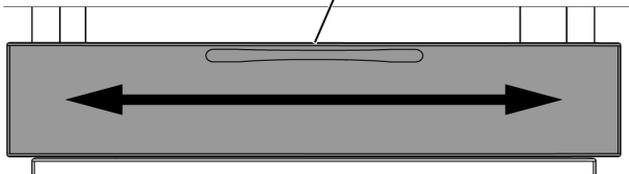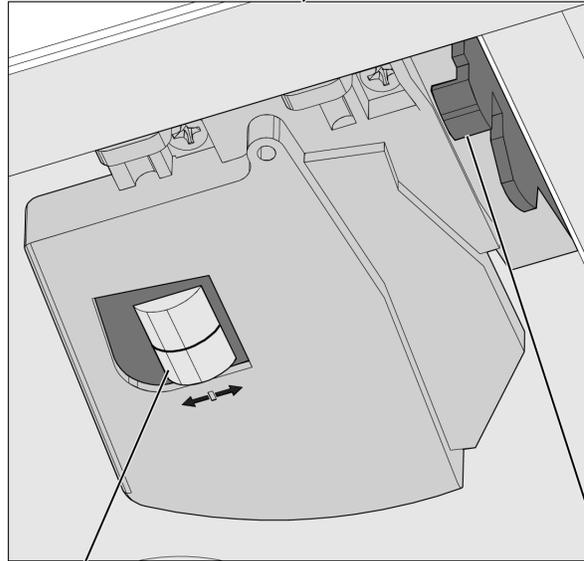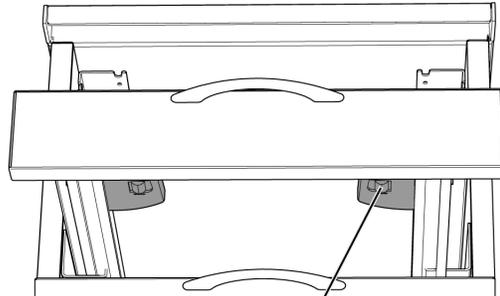

1 Side to Side: Rotate all barrels in the direction you want the drawer face to move.

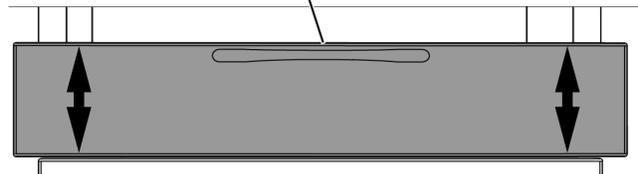

2 Up and Down: Slide the gray tab on the side to be moved. Slide the tab toward the back of the drawer to raise the drawer face.


All Other Drawers

Recommended Tool


- #2 Pozidriv® screwdriver



CAUTION To avoid damaging the screws, use a #2 Pozidriv® screwdriver to adjust the screws that move the drawer faces.

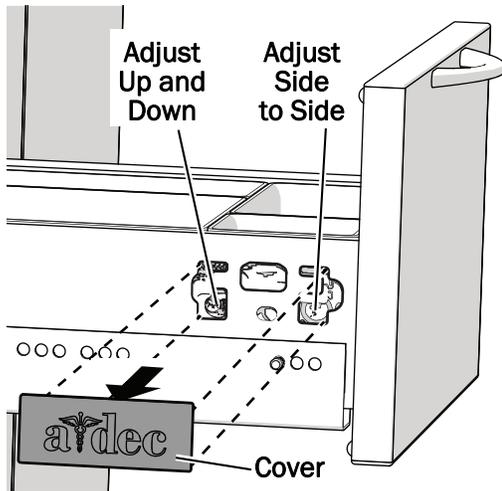


- 1** Remove the covers on both sides of the drawer.
- 2 Up and Down:** On both sides, adjust the screws closest to the back of the drawer.

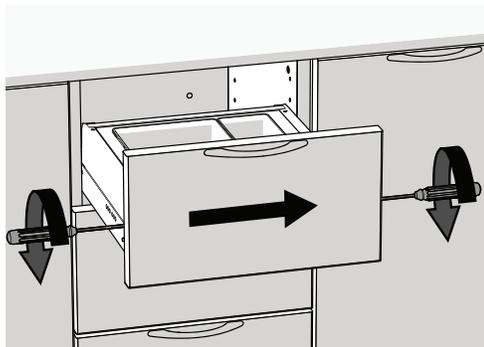

- 3 Side to Side:** On both sides, adjust the screws closest to the face of the drawer. Make sure to move both screws in equal increments. The direction shown moves the drawer face to the right.



TIP Use two screwdrivers at the same time to move the face more easily.

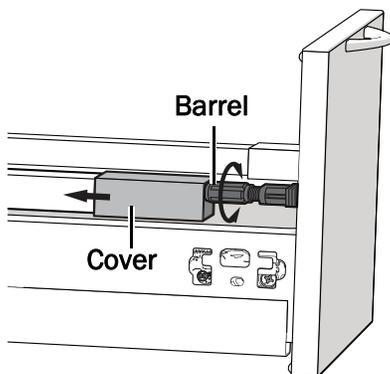

- 4 Front to Back Tilt (Gallery Rails only)** - Slide the covers and rotate the barrels.

Move the Cart



CAUTION Take care when maneuvering the cart over floor obstacles. A-dec carts are not intended to roll over thresholds or other floor obstacles. Secure the cart drawers and lift the cart high enough for the casters to move over any floor obstacles.

- 1** Use two hands on the top of the cart at all times when moving it, and ensure that the front is facing in the direction it's moved.

Regulatory Information and Warranty

For required regulatory information and the A-dec warranty, see the *Regulatory Information, Specifications, and Warranty* document (p/n 86.0221.00) available in the Resource Center at www.a-dec.com.



For quick access to this document online, scan, tap, or click this QR code, which points to a-dec.com/regulatory-guide.

Pozidriv is a registered trademark of the Phillips Screw Company in the United States and may also be a trademark or registered trademark in other countries. All other trademarks cited herein are registered trademarks or trademarks of A-dec, Inc. in the United States and may also be trademarks or registered trademarks in other countries.



A-dec Headquarters

2601 Crestview Drive
Newberg, OR 97132 USA
Tel: 1.800.547.1883 Within USA/CAN
Tel: +1.503.538.7478 Outside USA/CAN
www.a-dec.com

A-dec, Inc. makes no warranty of any kind with regard to the content in this document including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.



86.1103.00 Rev A
Date of Issue 2024-04-04
Copyright 2024 A-dec, Inc.
All rights reserved.
IGporeven20