



Recommended Monitor Mount Placement

The table and diagram below show recommended locations to place the A-dec Ceiling Monitor Mount. Primary recommendations minimize light and monitor arm interference.

IF THE SYSTEM HAS THIS TYPE OF LIGHT:	INSTALL THE MONITOR MOUNT HERE:	
	Primary Recommendation	Secondary Recommendation
Track-Mounted Ceiling Light	C2*	S1 or S2
Fixed-Mount Ceiling Light	C1	S1 or S2
Chair Mount Light	S1 or S2	C1
Preference-Mounted Light	S1 or S2	C1

* If installing at C2, position the monitor mount a minimum of 6-1/2" (165 mm) in front of the track light (towards 12 o'clock).

* NOTE: C2 position restricts comfortable use of monitor to only supine patient position.

A-dec Headquarters
 2601 Crestview Drive Newberg, Oregon 97132 USA
 Tel: 1.800.547.1883 Within USA /Canada
 Fax: 1.503.538.0276 Within USA /Canada
 Tel: 1.503.538.7478 Outside USA /Canada
 www.a-dec.com

A-dec 311A Dental Chair with Integrated Floor Box

Copyright 2021 A-dec Inc. All rights reserved.
 86.0912.01 Rev F

