

A-dec® Software Release Notes



Contents

| | | | |
|--|----------|--|----------|
| New Releases and Updates..... | 1 | Current and Past Releases | 2 |
| A-dec+™ Mobile and Web Applications..... | 1 | DS7 Software..... | 2 |
| | | CP5i/CP5 Software | 2 |
| | | A-dec+ Gateway Software | 3 |
| | | Universal Chair Board (UCB) Software | 3 |
| | | KaVo® Software | 3 |
| | | A-dec+ Mobile and Web Applications | 4 |
| | | A-dec+ Service Mobile Application | 4 |



NOTE Some features may not be available for all regions. If you have any issues or questions with your software updates, call A-dec customer service at 1.800.547.1883 (or +1.503.538.7478 outside the USA and Canada).

New Releases and Updates

A-dec+™ Mobile and Web Applications

Version 01.02.00 – released on 2024/04/18

- Added A-dec+ Cloud Connection Check to the **More** screen (iOS and Android only).
- Enhanced the **Dashboard** with a more accurate product counter and added the ability to organize products by room and location in **My Products**.
- Added **Connectivity ID** to the **General Information** table for A-dec 500 Pro delivery systems and A-dec+ Gateway products.
- Corrected identification of A-dec Simulator Pro products.
- Updated links to a-dec.com for customers in Australia and the United Kingdom.
- Added more informative product registration failure messages.
- A checkmark now appears when customer creates, resets, or changes an account password.
- Added a **Start Over** link to **Create Account** screens to easily return customer to the start of the process.
- Additional bug fixes and performance improvements.

Current and Past Releases

DS7 Software

Version 01.01.89 – released on 2024/03/15

- ▮ Resolved a low-voltage output issue when operating an analog ultrasonic scaler.

Version 01.01.82 – released on 2024/02/29

- ▮ Improved the UI so that the chair moves smoothly into position after a power cycle.
- ▮ Added the ability to select the chair's upholstery color in the UI.
- ▮ Removed the reverse icon from the Endo mode screen to eliminate confusion on icon function.
- ▮ **Dealer technicians only** – Resolved undesired behavior with X-Ray/Rinse preset position 4 while Enable/Disable mode was active.
- ▮ Additional bug fixes and performance improvements.

Version 01.01.48 – released on 2023/11/15

- ▮ Corrected an issue with the time zone resetting after software updates.
- ▮ Updated system to exit the screen saver when the delivery system handle is touched.
- ▮ Updated system to recall last active user when exiting the screen saver.
- ▮ Additional bug fixes and performance improvements.

Version 01.01.29 – released on 2023/09/12

- ▮ Corrected unresponsive chair position graphic.
- ▮ Corrected serial number deletion issues due to a factory reset.
- ▮ Corrected drive air pressure reading anomaly.
- ▮ Additional bug fixes and performance improvements.

Version 00.04.140 – released on 2023/06/22

- ▮ Initial release.

CP5i/CP5 Software

Version 1.1.27 – released on 2024/03/15

- ▮ Resolved a low-voltage output issue when operating an analog ultrasonic scaler.

Version 1.1.23 – released on 2024/02/29

- ▮ Bug fixes and performance improvements.

Version 1.1.12 – released on 2023/11/15

- ▮ Added compatibility for Decade®, Cascade®, and Performer® chairs.
- ▮ Additional bug fixes and performance improvements.

Version 1.1.8 – released on 2023/09/12

- ▮ Updated home screen for devices not connected to a chair.
- ▮ Additional bug fixes and performance improvements.

CP5i/CP5 Software *(cont.)*

Version 0.7.109 – released on 2023/06/22

- ▮ Initial release.

A-dec+ Gateway Software

Version 01.01.82 – released on 2024/02/29

- ▮ Bug fixes and performance improvements.

Version 01.01.48 – released on 2023/11/15

- ▮ Bug fixes and performance improvements.

Version 01.01.29 – released on 2023/09/12

- ▮ Reduced connection drops and improved device connectivity.
- ▮ Additional bug fixes and performance improvements.

Version 00.06.140 – released on 2023/06/22

- ▮ Initial release.

Universal Chair Board (UCB) Software

Version 1.1.23 – released on 2024/02/29

- ▮ Bug fixes and performance improvements.

Version 1.1.12 – released on 2023/11/15

- ▮ Bug fixes and performance improvements.

Version 1.1.8 – released on 2023/09/12

- ▮ Corrected cleaning position functionality for use with the A-dec Inspire® Control Center.
- ▮ Additional bug fixes and performance improvements.

Version 0.1.27 – released on 2023/06/22

- ▮ Initial release.

KaVo® Software

Version 1.3.23 – released on 2024/02/29

- ▮ Bug fixes and performance improvements.

Version 1.3.8 – released on 2023/11/15

- ▮ Bug fixes and performance improvements.

Version 1.3.4 – released on 2023/09/12

- ▮ Bug fixes and performance improvements.

Version 1.2.37 – released on 2023/06/22

- ▮ Initial release with A-dec 500 Pro and A-dec 300 Pro products.

A-dec+ Mobile and Web Applications

Version 01.01.00 – released on 2023/12/08

- ▮ Added a software update progress indicator to individual product screens (*previously, this only appeared on the top-level My Products screen*).
- ▮ Added ability to unregister products using the web app.
- ▮ Added ability for Group accounts to approve, deny, and manage clinic account link requests in the web app.
- ▮ Updated link functionality on the Dashboard page to My Products/My Clinics pages in the web app.
- ▮ Corrected issue with chair status when using Entry/Exit chair preset.
- ▮ Disabled third-party keyboards in the iOS® application as a general privacy precaution.
- ▮ Added support for Simulator Pro products.

Version 01.00.00 – released on 2023/06/22

- ▮ Initial release.

A-dec+ Service Mobile Application

Version 01.02.00 – released on 2023/12/06

- ▮ Removed parts pricing from the user interface (UI). The parts lists remain sorted by relative price (*USD, low to high*).
- ▮ Additional bug fixes and performance improvements.

Android® version 01.01.00 – released on 2023/10/12

iOS® version 01.01.01 – released on 2023/10/12

- ▮ Added reference tab with access to knowledge base and parts catalog.
- ▮ Improved login and registration.
- ▮ Improved search features.
- ▮ Expanded the number of parts discoverable via visual search.
- ▮ Additional bug fixes, UI refinements, and performance improvements.

Version 01.00.00 – released on 2023/06/22

- ▮ Initial release.



 **A-dec Headquarters**

2601 Crestview Drive
Newberg, Oregon 97132
United States

Tel: 1.800.547.1883 within USA/CAN
Tel: +1.503.538.7478 outside USA/CAN
www.a-dec.com

A-dec Australia

Tel: 1.800.225.010 within AUS
Tel: +61.(0).2.8332.4000 outside AUS

A-dec United Kingdom

Tel: 0800.ADEC.UK (2332.85) within UK
Tel: +44.(0).24.7635.0901 outside UK

In the United States, KaVo is a registered trademark of KaVo Dental GmbH, Android is a registered trademark of Google LLC, IOS is a registered trademark of Cisco Technology, Inc., and Safari is a registered trademark of Apple Inc. These may also be trademarks or registered trademarks in other countries. All other trademarks cited herein are trademarks or registered trademarks of A-dec, Inc. in the United States and may also be trademarks or registered trademarks in other countries.

86.1087.00 Rev H
Date of Issue 2024-04-18
Copyright 2024 A-dec, Inc.
All rights reserved.